

WHY SHOULD YOU TEST YOUR CIDEX® SOLUTION?

To ensure each CIDEX® Solution efficiently destroys 100% of *Mycobacterium tuberculosis*, the minimum effective concentration (MEC) required for high level disinfection should be tested with the appropriate CIDEX Solution Test Strip prior to each usage.

For optimal results, follow instructions for use carefully. Tightly re-cap test strip bottle after each use.

DISCARD ANY UNUSED TEST STRIPS 90 DAYS AFTER BOTTLE IS ORIGINALLY OPENED.

Cidex® OPA Solution Test Strips

WARNING: DO NOT USE CIDEX® OPA SOLUTION BEYOND 14 DAYS.


How to test: Completely submerge indicating pad of the strip into solution. Hold for 1 second and remove. Do not shake strip.

Remove excess solution by standing the strip upright on a paper towel.


Read results at 90 seconds. Do not read past 90 seconds. Pad will be completely PURPLE to indicate effective solution. IF ANY BLUE remains on the pad apart from the top line, solution is ineffective and must be discarded.



Pass

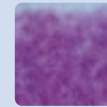

Fail

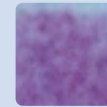

Fail



Fail

Reorder	Description	Case Contents
20392	CIDEX OPA Solution Test Strips	60 Strips/Bottle 2 Bottles
20393	CIDEX OPA Solution Test Strips	15 Strips/Bottle 2 Bottles

Cidex® Activated Dialdehyde Solution Test Strips

WARNING: DO NOT USE CIDEX® ACTIVATED DIALDEHYDE SOLUTION BEYOND 14 DAYS.


How to test: Completely submerge indicating pad of the strip into solution. Hold for 3 seconds and remove. Do not shake strip.

Remove excess solution by standing the strip upright on a paper towel.

Read results at 75 seconds. Do not read past 75 seconds. Pad will be completely PURPLE to indicate effective solution. IF ANY ORANGE remains on the pad apart from the top line, solution is ineffective and must be discarded.



Pass

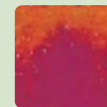

Fail



Fail



Fail

Reorder	Description	Case Contents
2920	CIDEX Solution Test Strips	60 Strips/Bottle 2 Bottles
2927	CIDEX Solution Test Strips	15 Strips/Bottle 2 Bottles


Cidexplus® Solution Test Strips

WARNING: DO NOT USE CIDEX PLUS® SOLUTION BEYOND 28 DAYS.


How to test: Completely submerge indicating pad of the strip into solution. Hold for 1 second and remove. Do not shake strip.

Remove excess solution by standing the strip upright on a paper towel.


Read results at 90 seconds. Do not read past 90 seconds. Pad will be completely RED to indicate effective solution. IF ANY YELLOW remains on the pad apart from the top line, solution is ineffective and must be discarded.

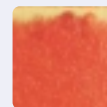

Pass

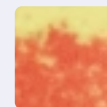

Fail

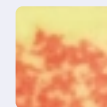

Fail



Fail

Reorder	Description	Case Contents
2924	CIDEX PLUS Solution Test Strips	60 Strips/Bottle 2 Bottles
2926	CIDEX PLUS Solution Test Strips	15 Strips/Bottle 2 Bottles